


### CONNECT AND SET UP THE DREAMSTATION S/T

#### 1 How do I connect the ventilator? (Image A)


- Connect device to the mains (AC) (Image B). by inserting power supply cord into socket on side of device 3 and plug the power supply 5. into mains 6.
- Connect 2 to the patient breathing circuit (flexible tubing).

- 1 Humidifier connector
- 2 Air outlet
- 3 Power Inlet
- 4 Door, Accessory access (SD card and filter access)

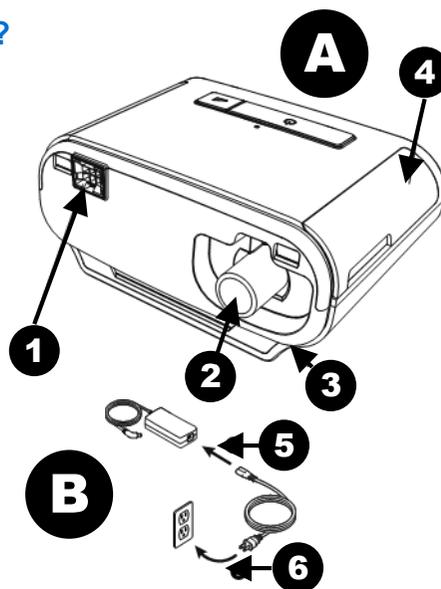

#### 2 How do I set up the breathing system and the HME filter? (Image C)

- Connect bacterial filter (HME) to air outlet 2
- Connect the flexible (patient circuit) tubing 7 to the outlet of the bacterial filter
- Attach facemask to tubing, marked at position 8
- Oxygen can be added to the patient circuit only if a pressure relief valve is used to prevent backflow of oxygen to machine
- Pressure relief valve 9 has inlet and outlet of 22mm, one side is attached directly to the air outlet 2 and then to the HME and subsequently the breathing circuit
- A bleed in valve is positioned in the circuit (Image D) and attached to the wall/ regulated cylinder supply with oxygen tubing (Image E)
- CPAP mask may have integrated oxygen inlet that connects to oxygen.

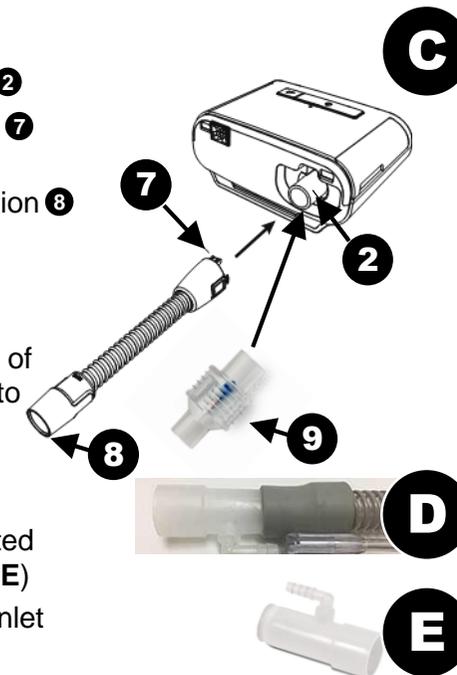


### SET THE CONTROLS ON THE DREAMSTATION S/T

#### 3 Navigation system for setting controls (Image F)

- 9 Therapy on/off button
- 11 Ramp button
- 12 LCD display screen
- 13 Control dial

**To quick start device**, press therapy button on/off button 9 this will initiate the airflow as set up by the user (Section 5) and needs to be set to CPAP. Delivered pressure will appear on LCD screen (Image J).

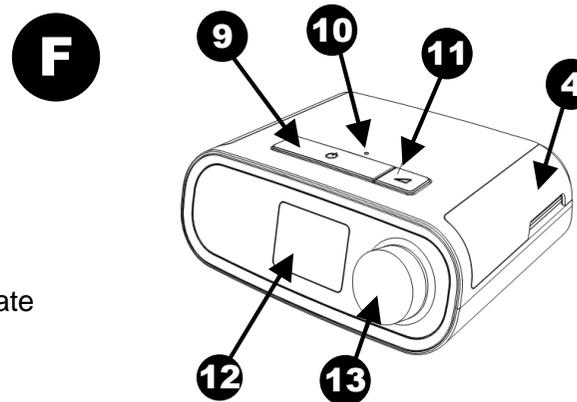


### OPERATE THE DREAMSTATION S/T

#### 4 How do I select the mode? (Image D)

- Turn device ON
- Press and hold both the control dial **13** and the ramp **11** button for 5 seconds to place in provider mode (Image H).


#### 5 How do I change settings? (Image H)

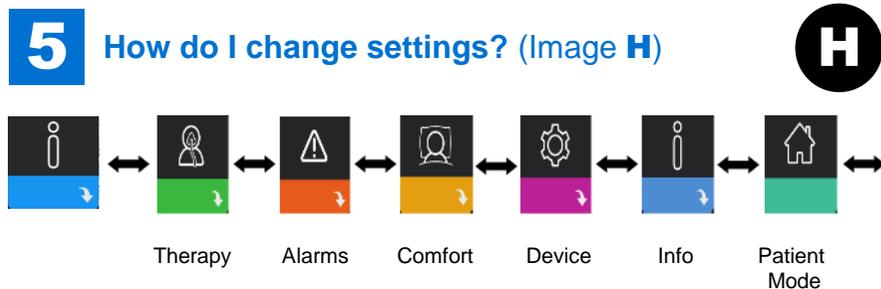

- In Provider mode Press the **Therapy** button
- In Therapy mode (Image H) select **CPAP** which will appear on the LCD screen **12**
- Press the control dial **13** to select that setting
- In CPAP mode adjust pressure (between 4 to 20cmH<sub>2</sub>O) confirming by pressing the control dial


#### 6 How do I check the device? (Image I/J)

Select 'My Info' (see table) (Image I)  
LCD Display screen (Image J)

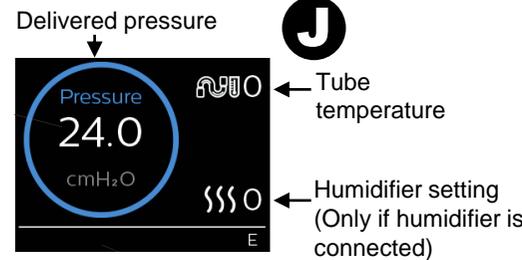

#### 7 What do the alarms mean? (Image K)

- Red flashing alarm (**high priority**). Audio pause **14** and control dial will flash **RED 13**
  - Yellow flashing alarm (**medium priority**). Audio pause **14** and control dial will flash yellow **13**
  - Alarms silenced by pressing the control dial **13**
- Alarm/alert occurs when:
- Device inoperable
  - Low minute ventilation
  - Apnea
  - Low tidal volume
  - Patient disconnection
  - Low/high pressure
  - Low motor speed
  - Power failure

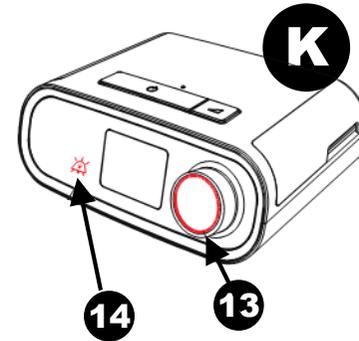

Icon	Text	Description
	Therapy hours	Amount of time user has received therapy.
<b>AHI</b>	AHI	Apnea/hyponea Index.
	Mask fit	Displays value "100% minus leak"
<b>Periodic Breathing</b>	Periodic Breathing	Displays the percentage of time that the user experienced periodic breathing.
<b>90% Pressure</b>	90% Pressure	Displays the nightly value of 90% Pressure



Latest training information can be found on: <https://portal.e-lfh.org.uk/Component/Details/606835>